

# CP-BU NEW

## Film



- Member of the CP family of Agfa HealthCare film
- High-speed, medium to high contrast bluesensitive film
- Suited for a complete range of applications

Blue-sensitive X-ray film  
designed for general radiology

UNIVERSAL FILM WITH A CONSTANT AND HIGH IMAGE QUALITY THANKS TO AGFA HEALTHCARE'S DISC GRAIN TECHNOLOGY.

CP-BU NEW film is a high-speed, medium- to highcontrast bluesensitive X-ray film that has been designed for use in general radiology in combination with blue-emitting rare earth and calcium tungstate intensifying screens.

Designed for a complete range of applications

CP-BU NEW film is a universal film because of the design of the characteristic curve. On the one hand, the contrast in the middle section and in the shoulder of the curve is sufficient to guarantee excellent visualization of all relevant details in applications, such as bone work and angiography. All lungfield structures are very clear presented as well.

On the other hand, the toe of the curve has been studied in such a way, that the information in low-density areas, such as the diaphragm, retrocardial, mediastinal and spinal areas are displayed very well too.

Always consistent images

CP-BU NEW excels in consistency: Agfa HealthCare's Disc Grain technology keeps image quality at a consistently high level, even when the film is processed in weaker chemicals or unfavorable conditions. CP-BU NEW also has a low sensitivity to desensitizing marks.

## Easy to read, nice neutral look


The use of a deep blue polyester base results in images with a nice, neutral look which are easier to read and more pleasant to look at.

## CP family of Agfa HealthCare films

The CP family of Agfa HealthCare films allows you to select the film which matches your specific requirements best.


- CP-BU NEW film - blue-sensitive universal film
  - CP-GL film - green-sensitive latitude film
  - CP-G Plus film - green-sensitive universal film
  - CP-GU film - green-sensitive high speed universal film
- Furthermore, specially designed screens are available for these films.

### Sensitometric curve

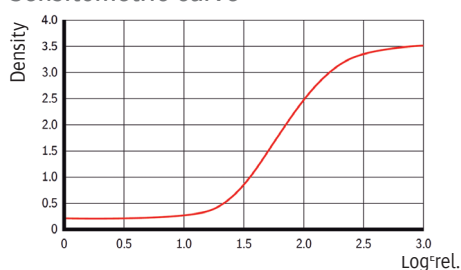

Seasoned premium developer, in an RP-system

### Spectral sensitivity curve

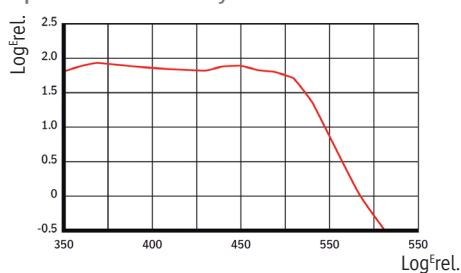

Sensitivity according to the energy necessary to attain a density of 1 above fog

## Technical

### SPECIFICATIONS

#### Safelight Requirements

- The use of a special red safelight filter in conjunction with a 15 watt frosted bulb is recommended
- It should be utilized and be positioned at least 1.2 m (4 feet) from the unloading/loading area and processor tray

#### Processing

- CP-BU NEW film is designed for RP, extended and manual processing
- Agfa HealthCare developer and fixer processing solutions are recommended

#### Handling

- The film should be handled with care
- Creasing and buckling as well as physical pressure should be avoided

#### Storage

- Unopened, unexposed film should be stored at or below 21 °C (70 °F) in a dry location
- The relative humidity is preferably between 30 and 50%
- The film should be protected from ionizing radiation

#### Packaging

- CP-BU NEW film is available in all customary sizes in 100 sheet packaging
- Other packaging and sizes are available on special order

For more information, please contact your local Agfa HealthCare sales representative

Agfa and the Agfa rhombus are trademarks of Agfa-Gevaert N.V., Belgium, or its affiliates. All other trademarks are held by their respective owners and are used in an editorial fashion with no intention of infringement. The data in this publication are for illustration purposes only and do not necessarily represent standards or specifications, which must be met by Agfa HealthCare. All information contained herein is intended for guidance purposes only, and characteristics of the products and services described in this publication can be changed at any time without notice. Products and services may not be available for your local area. Please contact your local sales representative at [agfa.com](http://agfa.com) for availability information. Agfa HealthCare diligently strives to provide as accurate information as possible, but shall not be responsible for any typographical error.

© Copyright 2010 Agfa HealthCare NV

All rights reserved

Printed in Belgium

Published by Agfa HealthCare NV

B-2640 Mortsel-Belgium

NE9A2 GB 00201005